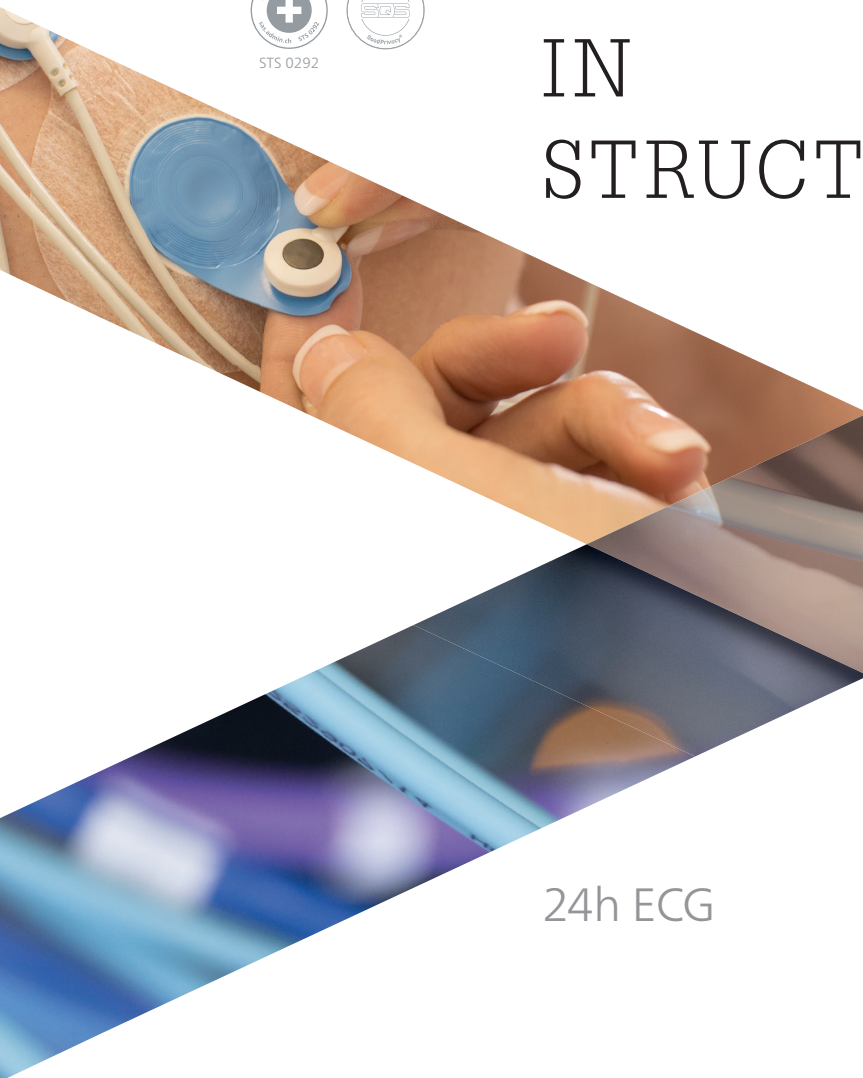
IN STRUCTION

24h ECG

VIOLLIER

Kardiologie – Pneumologie

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
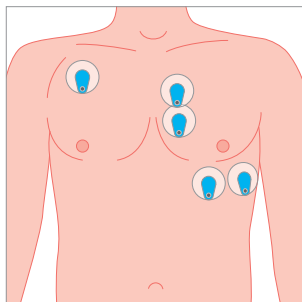
Attaching the monitor



Mounting the monitor

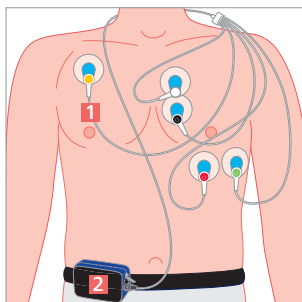- Degrease the skin using ethanol 70% and shave the hair for optimal adhesion of the electrodes.
- Position the self-adhesive electrodes as depicted in the diagram.
- Attach the cables with the correct colors as depicted in the diagram:
 - yellow
 - white
 - black
 - red
 - green
- Place the cable around the neck and feed it down the right side of the body → **Cables must not be glued 1.**
- Wear the **monitor 2** in the protective pouch on the belt around the waist.

Start monitor

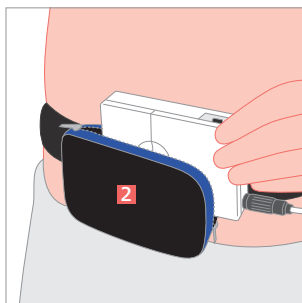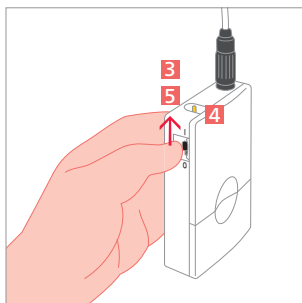- Tilt the black **sliding switch 3** to position I → **Do not switch off again.**
- Start recording: **Press and hold the yellow button 4 for 10 seconds** until a 3-beep tone confirms the start.
- After 24 hours switch off the monitor (position 0) **5**, dismount and return.

Accessories

- Monitor with plugged-in cable, chip card, and batteries inserted
- Protective pouch with adjustable belt
- Self-adhesive electrodes
- Order form, patient record, and instructions for use
- Red closure clip for the return

Return

After completing the recording, please return the monitor to us **with the cable plugged-in**, chip card, protective pouch, belt, order form, and patient record.

- by Customer service
- by mail

Please follow the instructions for the return of the box:

- Remove the label from the window, turn it over
→ **Viollier AG address is visible**
- Insert the label into window
- Close the box with the red closure clip
- Viollier will pay the postage