

Accessories

- Device with 4 cables
- Self-adhesive electrodes
- Order form, patient diary and instructions
- Red closure clip for return

Return

Please return the device, order form and patient diary to us at the end of the recording. Viollier will cover the shipping costs.


- By customer service – delivery in practice
- By DPD – drop off at a pickup parcelshop → dpgroup.com/ch/mydpd/parcel-shops
- By mail

Please note the following instructions for the return of the box:

- Remove label from window
- Slide the corresponding returns label (in the case) into the field of vision
→ **Viollier AG address becomes visible**
- Close case with red closure clip

Herz-Lungen-Praxis | Spalenring 147 | 4055 Basel | T +41 848 121 121 | viollier.ch
Monday – Friday 07:30 – 18:00 h

© by Viollier AG / 12.2025 / 18593

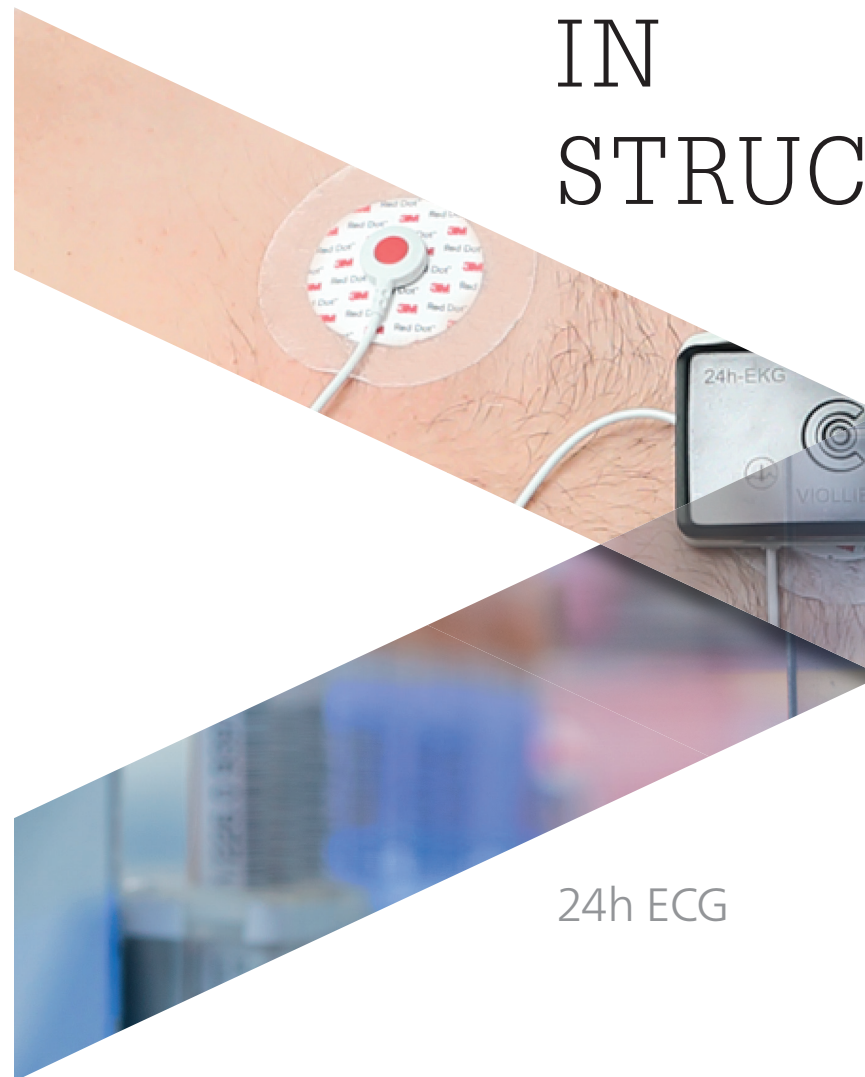

24h ECG



IN STRUCTION

Important information

Recording begins within 60 seconds of the electrodes making contact with the skin.

- Suitable for patients weighing 10 kg and over
- Do not use in the vicinity of CT, MRI and X-rays
- Mobile phones, strong electromagnetic sources and electric blankets can affect the quality of the recording.
- Wearing in the swimming pool, sauna, bathtub, shower or similar wet rooms is prohibited.

[illegible]

Patient diary

Please inform the patient about the diagnostic importance of keeping a correct and complete patient diary.

- Please use a black ballpoint pen.
- Note start and end of recording with date and time.
- Enter date and time of symptoms, unusual activities, medicines and sleeping periods.

[illegible]

Identification and analysis

- Patient's personal details
- The date you entered and the exact start time of the recording
- The monitor number entered by Viollier on the order form


Geräte-Nr.	006	Manschetten-Nr.
Chip-Nr.		Akku-Nr.

Please use the enclosed order form only, as this already has your address on it.

Attaching the monitor


Preparation

- Degrease the skin using ethanol 70% and shave the hair for optimal adhesion of the electrodes.
- Attach the electrodes to the cable ends.
- Attach an additional (any) disposable adhesive electrode to the push-button contact on the back of the device. This electrode is used to fix the device to the patient's chest. This reduces motion artifacts and improves the quality of the recording. This electrode has no medical / diagnostic function.

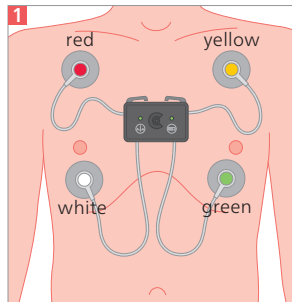

Monitor mount

- Position the self-adhesive electrodes on the patient's chest as shown on image **11**:
 ● red ● yellow ● green ○ white
- Press the edge of the electrode firmly against the skin to fix it in place.
- The device starts automatically within 60 seconds of the electrodes making contact with the skin.

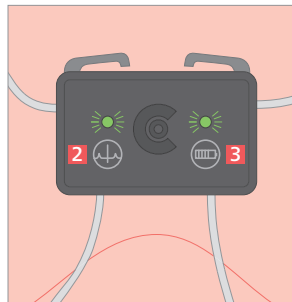

Check recording mode

- The 'QRS complex' symbol **2** indicates the recording status.
Green flashing light: ECG recording active
- The 'Battery' symbol **3** provides information on the charging status.
Green flashing light: battery voltage sufficient

By tapping the device firmly with the flat of the hand, the patient can set a marker in case of discomfort.