


¹³C respiratory test – Helicobacter pylori

¹³C respiratory test-set (56)

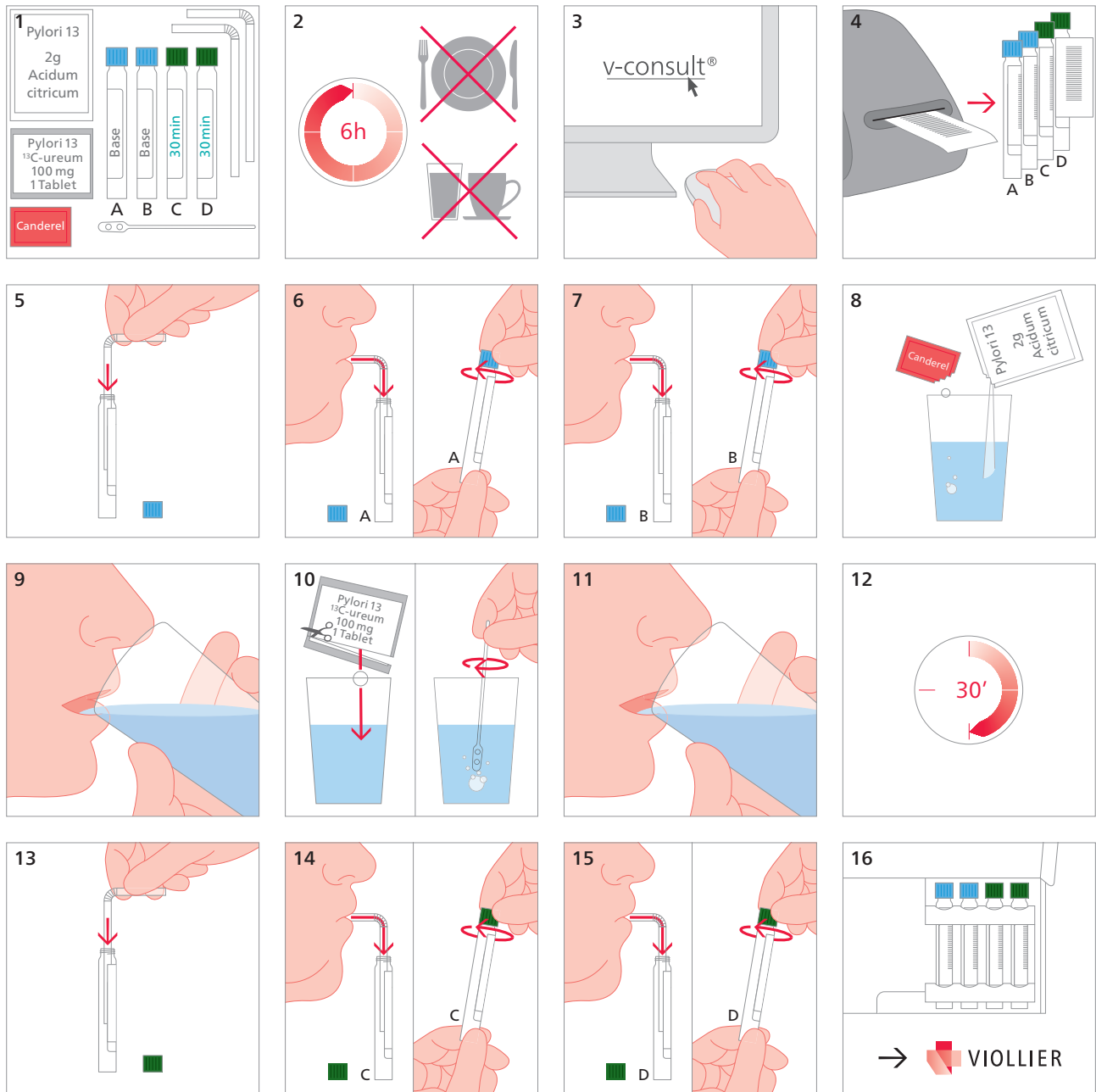

5 – 7 Deeply inhale, expire slowly and **completely (residual air)** blow through the straw into the blue tubes, close immediately

8 – 9 Dissolve 2 g citric acid (and, if desired, the sweetener tabs) in 100 mL H₂O and drink **completely**

10 – 11 Dissolve urea tablet (Urea ¹³C Tablet) in 100 mL H₂O and drink **completely**

12 **wait 30 minutes**

13 – 15 Deeply inhale, expire slowly and **completely (residual air)** blow through the straw into the green tubes, close immediately

Shipping material:

Please label with name, first names and date of birth of the patient