


Blood sampling – coagulation

Citrate tube, light blue (4)

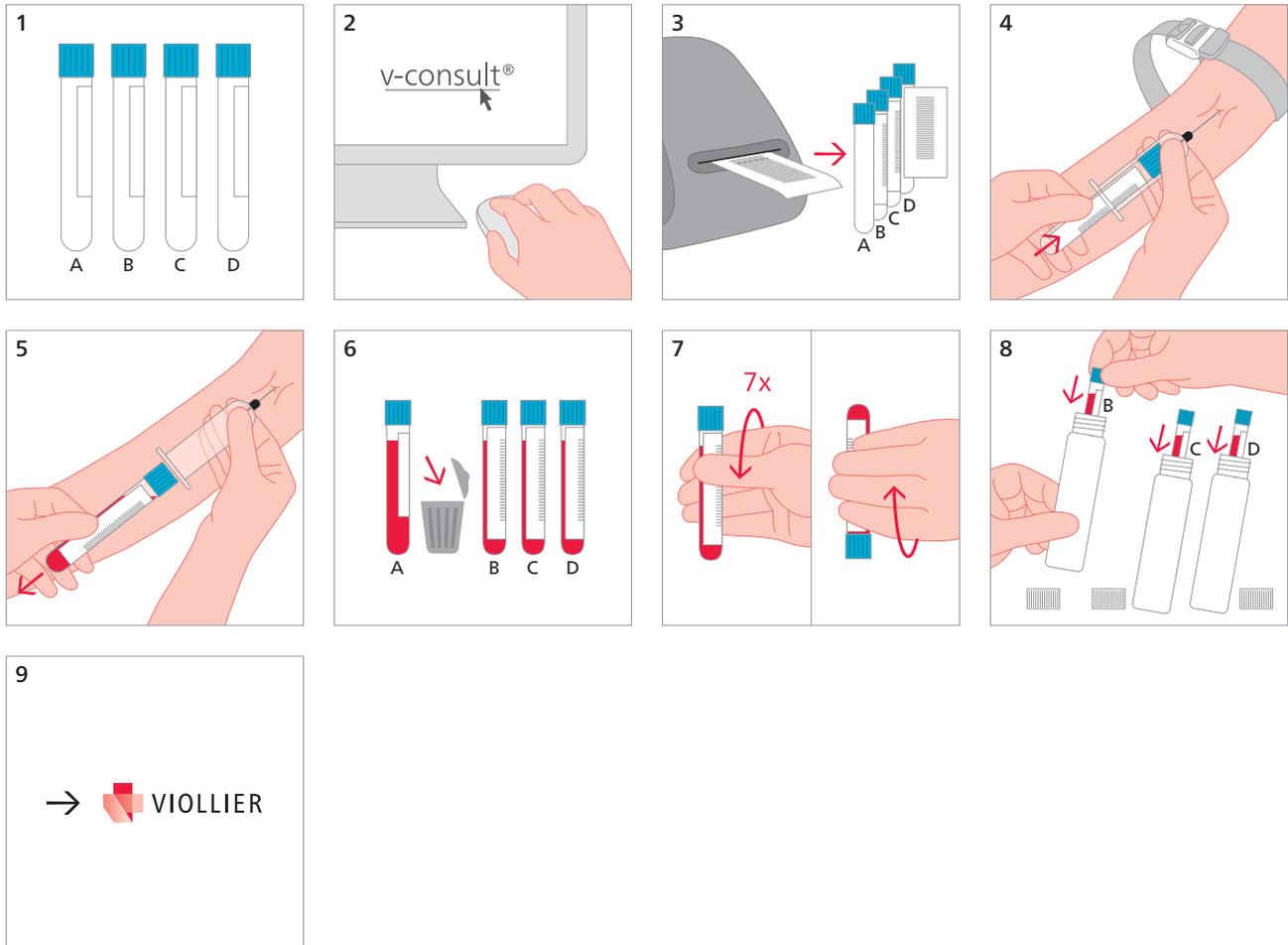

- | | |
|-------|---|
| 1 | Check expiry date |
| 4 | Constrict the blood flow with the tourniquet as little as possible or release the tourniquet immediately after penetrating the vein
Do not tap vein
Atraumatic venipuncture |
| 6 | Discard first tube (A)
Completely fill tube up to the mark (mixture ratio blood:citrate) |
| 8 – 9 | Transport uncentrifuged tubes immediately by courier
Special coagulation investigations: inquire by phone and define time of transport |

Shipping material:

Please label with name, first names and date of birth of the patient