


## Saliva – saliva collection

Salivette (74)

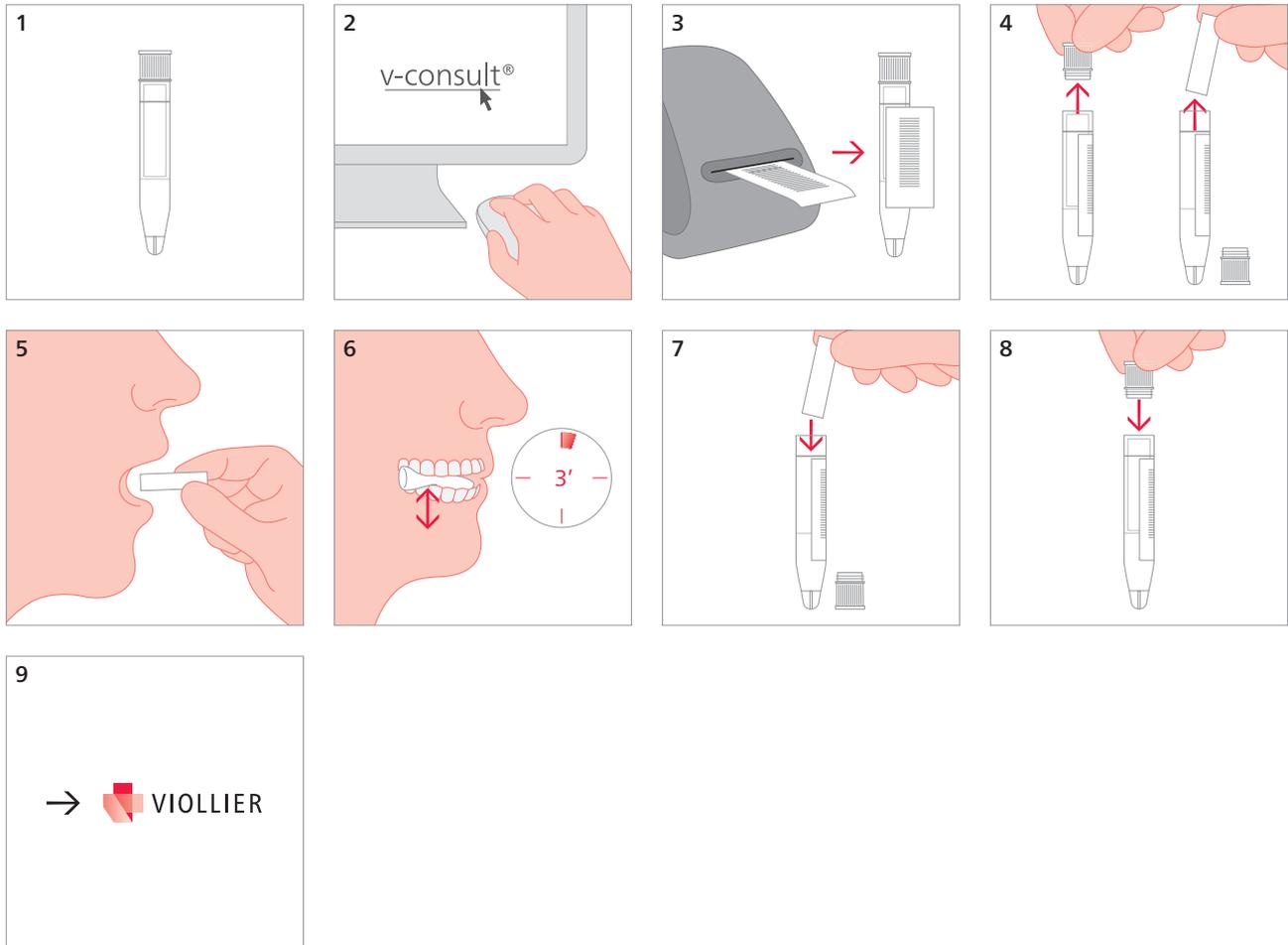

- |   |  |
|---|--|
| 4 | Remove cotton roll from Salivette  |
| 5 | Take the cotton roll in the mouth at the earliest 30 min after admission of food or drugs<br>(not for patients with increased risk of choking, not for children < 3 years) |
| 6 | Lightly chew cotton roll for 3 min   |
| 7 | Put saliva-saturated roll back into Salivette  |
| 8 | Firmly close Salivette   |

Shipping material:

Please label with name, first names and date of birth of the patient